

HUG A SHOULDER.



CRADLE A HIP.



UPLIFT A SOLE.



INTRODUCING THREE MATTRESS OVERLAYS

DESIGNED TO SUPPORT THE BODY

Protection. Comfort. Help in healing. These are the benefits

AND LIFT THE HUMAN SPIRIT.

you've come to expect from a mattress support system. Even so,

maybe it's time you expected something more. Introducing three

mattress overlays from The ROHO Group – PRODIGY, SOFFLEX and

the ROHO DRY FLOATATION Mattress Overlay – that are advancing

patient care to a new, more personal, level.



YOU'LL NEVER THINK THE SAME WAY AGAIN.

Although designed for different patient applications, all three mattress support systems share common features that go hand-in-hand with their unique air floatation technology. It is this technology, featuring soft, flexible interconnected air cells, that allows for each support system to provide a personal, customized fit for each and every patient:

- Zoned / Modular - Each section can be adjusted to address the specific needs of the patient's body shape and size.
- Adjustable - Air cells can be adjusted to shape and conform to the patient's body.
- Non-powered - No external power source is required.
- Lightweight - Lightweight material makes it easy to transfer system to different care settings.

Linked by technology, yet designed for unique patient applications, PRODIGY, SOFFLEX and the ROHO DRY FLOATATION Mattress Overlay make for an ideal family of mattress support systems to provide care from prevention to treatment for your patients.

THE PRODIGY MATTRESS OVERLAY®



Portable: Easy-to-use design appropriate for all care settings; thereby ensuring continuity of care.

Zoned: The PRODIGY Mattress Overlay comes in three sections. Each section can be adjusted to address the specific needs of your patient's body shape and size.

Adjustable: The 6.5" x 7.5" x 3.5" (16.5 cm x 17.1 cm x 9 cm) air cells can be adjusted to shape and conform to your patient's body providing a total contact environment.

Non-powered: The PRODIGY Mattress Overlay requires no external power source in order to operate effectively; therefore, no energy consumption or worries associated with pump or blower failure.

Lightweight: The PRODIGY Mattress Overlay weighs two (2) pounds per section (.9 kg) or six (6) pounds (2.7 kg) for a complete system.

Durable Polyvinyl Material: The PRODIGY Mattress Overlay is constructed of durable polyvinyl and can easily be cleaned with soap and water.

Drainage Holes: Drainage holes are placed in the PRODIGY Mattress Overlay to pull fluids away from the incontinent patient, and to address the problems of maceration associated with wound healing.

12-Month Limited Warranty

Construction - Flame resistant polyvinyl

Weight - Six (6) pounds (2.7 kg) for a complete system

Size - A complete three unit system equals 81.5" x 35.5" (207 cm x 90.2 cm) and fits a standard size hospital or homecare bed (27" x 35" / 68 cm x 89 cm per section)

Cleaning - Clean with soap and water

Single patient use

ADDING A PERSONAL TOUCH. THE PRODIGY MATTRESS OVERLAY.

You would like to provide your patients with personalized care and attention, even in the most common of patient care circumstances. The ROHO Group allows you this option by providing you with a uniquely designed, overlay that outperforms other products in its class. Through a non-powered, multi-zoned, air-adjustable approach to wound care, the PRODIGY Mattress Overlay provides a unique solution to those patients who are at-risk of skin breakdown and/or have a Stage I or II ischemic ulcer. PRODIGY's interconnected air-filled cells conform to the unique body shape and size of the patient, providing a total contact environment. Plus the three section PRODIGY is so easy to set-up, inflate and adjust, it is the ideal choice for protection in any care setting.

THE SOFFLEX® MATTRESS OVERLAY



Zoned: The SOFFLEX Mattress Overlay comes in three sections. Each section can be adjusted to address the specific support surface needs of your patient's distinct body shape and size.

Adjustable: The 4" x 4" x 3.25" (10 cm x 10 cm x 8 cm) air cells can be adjusted to shape and conform to your patient's body providing a total contact environment ideal for facilitating wound healing.

Non-Powered: The SOFFLEX Mattress Overlay requires no external power source in order to operate effectively; therefore, no energy consumption or worries associated with pump or blower failure.

Lightweight: The SOFFLEX Mattress weighs three (3) pounds per section (1.4 kg) or nine (9) pounds (4 kg) for a complete system.

Durable Polyurethane Material: The SOFFLEX Mattress is constructed of durable, flame retardant polyurethane, and can be easily cleaned with soap and water.

Drainage Holes: Drainage holes are placed in the SOFFLEX Mattress to pull fluids away from the incontinent patient, and to address the problems of maceration associated with wound healing.

12 month limited Warranty

Construction-Flame retardant polyurethane

Weight-Nine (9) pounds (4 kg) for a complete system

Size-A complete three unit system equals 80" x 36" (203 cm x 91 cm) and fits a standard size hospital or homecare bed (27" x 36" per section / 69 cm x 91 cm)

Cleaning-Clean with soap and water

Covers Available

GETTING TO THE HEART OF COMFORT. THE SOFFLEX MATTRESS OVERLAY.

Most clinicians go out of their way to make their patients comfortable. Comfort, however, doesn't have to be limited to just the patient. SOFFLEX was designed for the comfort of the patient and the clinician as well.

For the clinician, SOFFLEX provides comfort in terms of a confidence level. Simple to use, and adaptable from hospital to home, SOFFLEX provides a cost-effective, therapeutic alternative for the ultimate in patient care.

For the patient, SOFFLEX creates an environment where they can comfortably lie, for extended periods of time, without the fear of compromised blood flow and tissue breakdown. Plus, SOFFLEX is zoned and fully adjustable, providing customized pressure relief.

THE ROHO[®] DRY FLOATATION[®] MATTRESS OVERLAY



Zoned: The ROHO Mattress Overlay comes in four sections. Each section can be adjusted to address the specific support surface needs of your patient's distinct body shape and size.

Adjustable: The three and one half (3.5) inch air cells can be adjusted to shape and conform to your patient's body and provide a total contact environment ideal for facilitating wound healing.

Non-Powered: The ROHO Mattress Overlay requires no external power source in order to operate effectively; therefore, no energy consumption or worries associated with pump or blower failure.

Lightweight: The ROHO Mattress weighs eight (8) pounds per section (3.6 kg) or thirty-two (32) pounds for a complete system (14.4 kg).

Durable Neoprene Rubber: Flame resistant and resistant to urine, oils and medicines. Easily cleaned with soap or water and can be steam autoclaved. Product can be repaired extending the overall life of the mattress.

Drainage Holes: Drainage holes are placed in the base to pull fluids away from the highly incontinent patient, and to address the problems of maceration associated with wound healing.

24 month limited Warranty

Construction-Flame resistant neoprene rubber

Weight- 32 pounds (14.4 kg) for a complete system

Size-A complete four unit system equals 74" x 34"

(188 cm x 86 cm) and fits a standard size hospital bed

Cleaning-Clean with soap and water, or can be steam autoclaved

Covers Available

HEALING WHEN IT'S NEEDED MOST. THE ROHO DRY FLOATATION MATTRESS OVERLAY.

If treating a critical care patient automatically brings to mind images of expensive, powered support surfaces, the ROHO DRY FLOATATION Mattress is worthy of some attention. The original, non-powered, adjustable, zoned, pressure-reducing ROHO DRY FLOATATION Mattress Overlay is therapeutic, cost-effective technology at its best. Hundreds of interconnected air cells allow a patient with any stage ischemic ulcer and/or myocutaneous flap to benefit from a custom-fitted healing environment. And weighing just thirty-two pounds (or 14.4 kg), it's completely portable and adaptable to numerous care settings.

No more worries associated with the more expensive, cumbersome, powered alternatives. The ROHO DRY FLOATATION Mattress Overlay keeps the focus right where it should be – on the patient.

MATRESS COMPARISON CHART

PRODUCT → FEATURE ↘	PRODIGY MATTRESS OVERLAY	SOFFLEX MATTRESS OVERLAY	ROHO DRY FLOATATION MATTRESS OVERLAY
APPLICATIONS:	For patients who are suffering from Stage I or II ischemic ulcers and have had a history of skin breakdown, and for patients who require comfort and protection	Can be used on patients with an ischemic ulcer up to and including stage III ischemic ulcers if edema not present.	Can be used on patients with any stage ischemic ulcer, myocutaneous flaps, skin grafts or burns.
ZONES:	Three (3) sections	Three (3) sections	Four (4) sections
ADJUSTABLE:	Yes	Yes	Yes
CONSTRUCTION:	Flame -resistant Polyvinyl	Flame-retardant Polyurethane	Flame-resistant neoprene rubber
WEIGHT:	2 lbs. per section (.9 kg) / 6 lbs. complete system (2.7 kg)	3 lbs. per section (1.4 kg) / 9 lbs. complete system (4 kg)	8 lbs. per section (3.6 kg) / 32 lbs. complete system (14.4 kg)
WEIGHT LIMIT:	300 lbs.	300 lbs.	No weight limit.
SIZE:	A complete three unit system equals 81.5" x 35.5" (207 cm x 90.20 cm).	A complete three unit system equals 80" x 36" (203 cm x 91 cm).	A complete four unit system equals 74" x 34" (188 cm x 86 cm). A complete three unit system equals 80" x 35" (203 cm x 89 cm).
CLEANING:	Clean with soap and water	Clean with soap and water	Clean with soap and water, or can be steam autoclaved
ACCESSORIES:	Comes complete with hand inflation pump, repair kit and operating instructions	Comes complete with hand inflation pump, repair kit and operating instructions. (covers available)	Comes complete with hand inflation pump, repair kit and operating instructions. (covers available)
NON-POWERED:	Yes	Yes	Yes
REPAIRABILITY:	Patch kits provided.	Patch kits provided	Yes. Patch kits also provided.
WARRANTY:	12-month limited	12-month limited	24-month limited
U.S. MEDICARE CODE:	E0197		E0371




THREE MATTRESS OVERLAYS DESERVING OF INDIVIDUAL ATTENTION.

Together, PRODIGY, SOFFLEX and the ROHO DRY FLOATATION Mattress Overlay are a complete family of mattress support systems. All three systems are zoned, adjustable, non-powered and lightweight, features that have proven advantages for any patient, in any care setting. Yet individually, these three mattress systems also cover a range of patient applications, from the at-risk patient to those who need treatment for the most advanced stage pressure ulcers.

Hug a shoulder. Cradle a hip. Uplift a sole.

With the PRODIGY, SOFFLEX and ROHO DRY FLOATATION Mattress Overlay on your side, you'll never think the same way again about support systems.

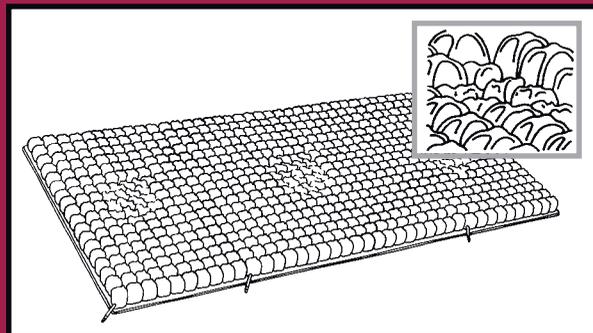

CONTOURING

When the patient transfers from the ROHO Mattress it may look as though the system has lost air or is not inflated enough for the patient. **DO NOT ADD AIR.** The above diagram is an example of the immersion and contouring of the ROHO Mattress to the patient's body shape and size*. When properly adjusted the patient is immersed **into** the ROHO Mattress... Not lying on top of the air cells.

*Contouring may not be apparent with extremely thin patients.

YOU'LL NEVER THINK THE SAME WAY AGAIN.

The PRODIGY, SOFFLEX, and ROHO DRY FLOATATION Mattress Overlays are produced with the highest level of quality and workmanship standards. Supporting the reliability and effectiveness of the product is a network of service providers dedicated to prompt, efficient, and understanding service, unparalleled in the industry.

For more information on how your tissue viability problems can be solved with The ROHO Group, speak with your service provider, or call us directly to speak with one of our regional support surface specialists. If you require supporting technical information and/or case studies involving *any of our mattress systems*, we'll be happy to supply you the information you need.

100 North Florida Avenue

Belleville, IL 62221-5429, USA

www.therohogroup.com

U.S. & Canada 1.800.851.3449 • 1.888.551.3449 (fax)

Outside the U.S. & Canada: 1.618.277.9150 • 1.618.277.6518 (fax)



The ROHO Group, Inc. has a policy of continuous product improvement and reserves the right to amend specifications presented in this brochure.